

**Ur24T™ F15**

Urine Collection Apparatus



**Female Instructions**

# Ur24T™ F15 Urine Collection Apparatus

The Ur24T™ F15 is an alternative to using a traditional internal catheter

## WHAT YOU NEED

1. A disposable Urine Collection Apparatus (UCA)
2. An aspirator pump. Recommended pump is Drive/DeVilbiss Vacu-Aide® Compact Suction Unit #7310PR or Stationary Aspirator #18600

## START THE PROCESS


The UCA is applied to the outside of the urethra and allows urine to be voided from the bladder without any discomfort

1. Wash your hands with soap and water
2. Clean the perineal area prior to applying the UCA
3. Read the user's manual for the recommended aspirator, manufactured by Drive/DeVilbiss
4. Connect the hose to the aspirator nozzle
5. Separate legs and labia (Figure 1)
6. Turn on the aspirator and check the UCA tip for suction
7. Place the UCA over the urethral opening (the hole where urine leaves the body). Urine will start to flow into the collecting canister (Figure 2)
8. Set the aspirator and adjust the setting for optimum urine output and patient comfort


## REMOVE UCA

Wait at least 30 seconds after the urine flow stops before removing


1. Separate legs and labia
2. Gently pull UCA directly outward (Figure 3). If urine collects in UCA, keep opening tilted upward when removing
3. Turn off the aspirator
4. Dispose of the urine and wash the canister
5. Wash the UCA with soap and water or the suggested Ur24T™ anti-bacterial solution and let dry
6. Empty the bladder every 4-6 hours, or as often as your healthcare professional recommends
7. If asked to keep track of your urine output, the negative pressure canister will help you measure. Take note of your urine amount, date and time

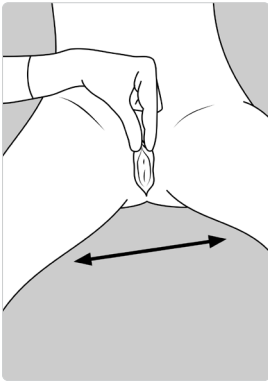

**Figure 1**  
Separate legs  
and labia

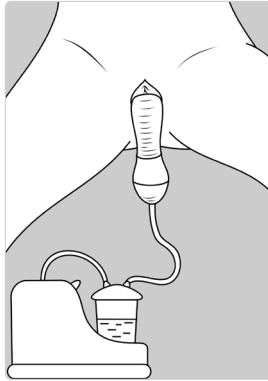

**Figure 2**  
Place UCA and wait  
while urine flows  
into canister

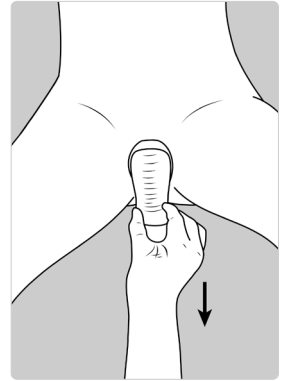

**Figure 3**  
Gently pull UCA  
outward to remove

# IMPORTANT

## Dispose of the UCA if it:

- ✗ Looks damaged, broken, or has holes
- ✗ Feels different than usual (hard, brittle, or other different texture)
- ✗ Stops draining

## Please note:

If the UCA is exposed to extreme temperatures, it may change color. This will NOT affect functionality.

## Safety information for hazards that may cause injury

1. Use this product only for its intended use as described in this guide
2. Be sure to use precaution to keep UCA clean
3. Do not use while bathing
4. Do not place in or drop into water or other liquid
5. Consult a physician in case of any allergic reaction when using the UCA. Note: All materials used in the UCA are latex-free
6. Keep the power cord and UCA away from heated surfaces
7. Keep UCA out of the reach of children and pets and away from pests
8. Don't let children play with the tubing as this may lead to strangulation

## Troubleshooting

Problem	Possible Cause(s)
<b>Inadequate Suction</b>	<ul style="list-style-type: none"><li>• Canister lid not on tight</li><li>• Tubing not connected properly</li><li>• Adjust the suction level</li><li>• Tubing or catheter clogged</li></ul>
<b>Unit does not turn on</b>	<ul style="list-style-type: none"><li>• Check power cord and outlet</li></ul>

**Indication for Use:** The Ur24Technology External Catheter is intended for non-invasive urine collection management in female and male patients.

The UCA is meant to be used just by you. Do not use the same UCA on more than one person. Do not recycle. If you experience any discomfort or problems, call your prescribing physician. DISPOSE OF ALL COMPONENTS AS INFECTIOUS WASTE.

**Please view instructional video before using the UCA.** Scan this code with your phone's camera, or visit [ur24technology.com/demo](http://ur24technology.com/demo)



# Ur24Technology.com

**Questions? Please contact Customer Service at 833-44-UR24T**  
2021-01-F-Rev. A • U.S. Patent Applications Number 16/881601  
Filed: 5/22/20 and Number 15/416272, Filed: 1/26/17  
Class 1 Exempt Device, 21 C.F.R. § 876.5250